

PERMEADERM

Bioactive Matrix Dressing



PermeaDerm bioactive matrix is a transparent, adherent, flexible dressing that provides a moist wound healing environment on cleanly debrided wounds when applied after hemostasis has been achieved. PermeaDerm enables the user to vary porosity, allowing moisture to pass to a secondary dressing that can be changed without interrupting the healing process.

INDICATIONS FOR USE

Permeaderm Sheets:

- Partial thickness burn wounds
- Donor sites
- Coverage of meshed autografts

Permeaderm Gloves:

- Coverage of debrided partial thickness hand burns

FEATURES & BENEFITS

- Adherence - flexible silicone-coated bioactive matrix allows for optimal wound contact and early mobilization once adherence is achieved
- Transparency - allows for visual wound assessment
- Moisture management - porosity can be varied by applying different amounts of stretch
- One-time application - reduces OR time and the need for multiple dressing changes
- Unique sizes - Multiple glove sizes and large sheets available

INSTRUCTIONS FOR USE

Application:

- Clean and debride wound
- Achieve hemostasis
- Stretch PermeaDerm to remove wrinkles and achieve desired porosity (optimal stretch 10%)
- Place PermeaDerm on wound with "This Side Up" label facing away from the wound
- Secure PermeaDerm using standard methods (e.g. steristrips, skin adhesive, roll gauze, etc.)
- Apply a non-adherent contact layer directly to PermeaDerm
- Apply an absorbent secondary dressing over the contact layer and secure using standard methods
- In the first 24 hours, maintain securement and do not get Permeaderm wet
- Change secondary dressings without disturbing PermeaDerm if necessary
- Permeaderm should be left in place until healing occurs.

Removal:

- Gently remove when the tissue underneath has healed - moistening Permeaderm prior to removal may help



PRODUCT INFORMATION

Size	Area	Code	Packaging
PermeaDerm 5" x 10" (12.7cm x 25.4cm)	322.6 cm ²	3000124725	8 ea/box
PermeaDerm 10" x 15" (25.4cm x 38.1cm)	967.7 cm ²	3000124724	3 ea/box
PermeaDerm 15" x 30" (38.1cm x 76.2cm)	2,903.2 cm ²	3000124726	1 ea/box
PermeaDerm Glove Extra Small	N/A	3000124727	2 ea/box
PermeaDerm Glove Small	N/A	3000124728	2 ea/box
PermeaDerm Glove Medium	N/A	3000124729	2 ea/box
PermeaDerm Glove Large	N/A	3000124730	2 ea/box
PermeaDerm Glove Extra Large	N/A	3000124741	2 ea/box

medproducts@milliken.com | healthcare.milliken.com | 1-866-491-6556