



PetFlex[®] AFD

ABSORBENT FOAM DRESSING

ANDOVER

by Milliken

All-in-one wound management system for puncture wounds, abrasions and lacerations.

Foam pad & PetFlex cohesive in one | Absorbs the equivalent of 29 gauze pads | Quick, easy application
Controlled compression -will not constrict | Won't stick to wound bed

How to Use



After cleaning the wound, unroll to expose absorbent foam pad.



Place foam pad over wound.



Cover wound with foam pad & wrap bandage around so it covers first layer. Prior to each wrap around leg, pull bandage out approx. 6".



Continue to wrap at desired tension with 50% overlap.



Seal end of bandage to bottom layer with finger pressure for a strong, solid seal.

Ordering Information:

Catalog #	Size	Rolls/Case
2220PW-016	2" x 2.5 yds.	16
2440PW-008	4" x 2.5 yds.	8

